



Snoreka Bite Card

This Bite Card will help you decide which Snoreka device will be right for you.

→ **Print**

Print this A4 PDF at 100%
(Double sided if possible)

→ **Cut out**

Cut along the bright pink line.

→ **Fold**

Fold on the dotted line
& hold the tab at the bottom.

→ **Put in mouth / Bite**

Put curved section into your mouth. Bite down with front teeth at dotted line & leave impression.

→ **Your bite size**

Teeth in Blue
= you need Snoreka Mini

Outside Blue
= you need Snoreka



SNOREKA



SNOREKA MINI

